



SELF-SEALING STERILIZATION POUCH

The medical adhesive tape can make rapid and efficient seal without any professional sealing devices.

PRODUCT DESCRIPTION

Material: French Arjo Wiggins(Sweden Billerud) 60g/m²Medical high-temperature dialysis paper + blue(white/green) medical CPP/PET compound film + medical adhesive tape

Features

- The medical adhesive tape can make rapid and efficient seal without any professional sealing devices

- The indicator's color has significant change and no change back; it will turn black from initial blue under steam sterilization and brown from initial pink under the EO sterilization;
- The three sided explosion-proof can efficiently prevent the pouch from bursting;
- The transparent compound film can show what inside the pouch clearly in order to reduce errors

How to use:

- Apply to hospital, clinic and laboratory's sterilization; and also apply to the disinfection of beauty products of family high-temperature;
- Apply to both ETO sterilization and STEAM sterilization
- The sharp part of equipment should be put contrary to the peel side to ensure safety use.
- Tear off the release paper aside the tangent line, seal the pouch and mark the sterilization time, contents and usage etc.
- The clear area with temperature bellow 25°C and humidity below 60% is recommended, the valid period will be 6 months after sterilizing.

CODE	SIZE	CODE	SIZE
1	57mm*130mm	7	200mm*320mm
2	70mm*260mm	8	200mm*360mm
3	90mm*165mm	9	230mm*395mm
4	90mm*260mm	10	305mm*430mm
5	135mm*280mm	11	400mm*535mm
6	190mm*360mm	12	500mm*630mm